

PRIMING AND LOADING THE DISPOSABLE SET



THESE INSTRUCTIONS ARE NOT A SUBSTITUTE FOR THE OPERATOR'S MANUAL.

Be sure to read the operator's manual thoroughly prior to operating the pump. See other side for pump operation instructions.

STEP 1


- If you are using the SpikeRight[®] PLUS set, remove protective cover from connector and insert connector tip into compatible port on the enteral formula container. Secure the connector by tightly screwing threaded components together.

- If you are using the 500 mL or 1200 mL disposable set, hold bag upright and pour in feeding solution. Close cap securely and hang bag.

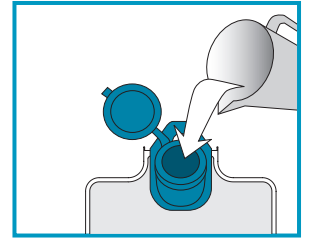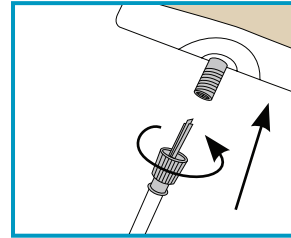



STEP 2

Remove protective cover from the distal end connector.

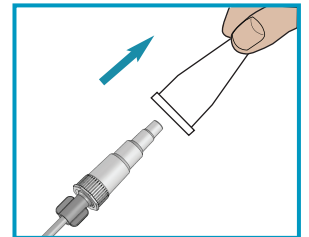

STEP 3 – PUMP PRIMING





Loop silicone tubing around pump wheel stretching lightly. Seat cassette into pump. Close pump door. Press and hold **ON/OFF** key for 1.5 seconds to turn pump on. Press and hold the **PRIME** key until formula reaches the distal end connector.

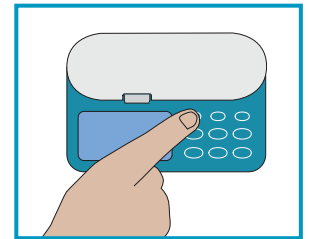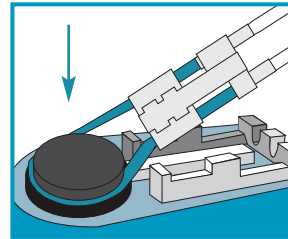

STEP 3 – MANUAL PRIMING

Gently pinch teal colored tubing below the "💧" symbol and squeeze bag at the same time to assist fluid flow. Hold this position until formula reaches the distal end connector. Loop silicone tubing around pump wheel stretching lightly. Seat cassette into pump. Close pump door.

For backpack use: Turn bag upside down and gently squeeze. Tilt bag as needed to keep air at tubing port. At the same time, gently pinch teal colored tubing below the "💧" symbol. Hold this position until air is removed from the tubing. Gently squeeze bag at the same time to assist fluid flow.

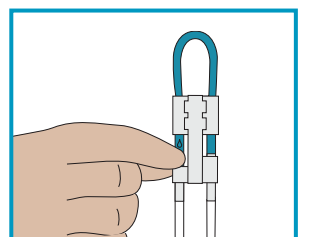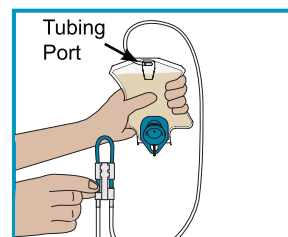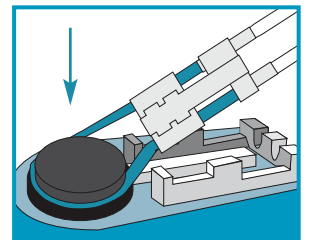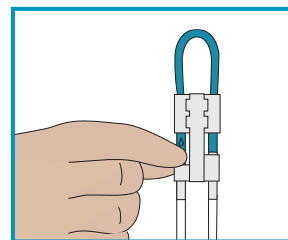

PROGRAMMING AND RUNNING THE PUMP


THESE INSTRUCTIONS ARE NOT A SUBSTITUTE FOR THE OPERATOR'S MANUAL.

Be sure to read the operator's manual thoroughly prior to operating the pump.

See other side for delivery set priming and loading instructions.

STEP 1

Hang feeding container so that the bottom of the bag is at or above the level of the pump door.

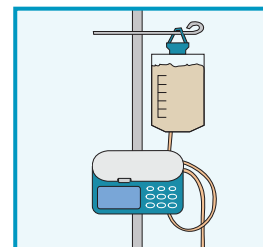


STEP 2

Press and hold **ON/OFF** key for 1.5 seconds to turn pump on. After running the self test, display will show last programmed rate.




STEP 3 – Program Rate

Program the prescribed rate using the **(+)** and **(-)** keys.

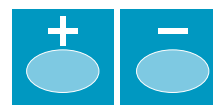


STEP 4 – Dose Feeding

Press **RATE/DOSE** key to display dose. Program the prescribed dose using the **(+)** and **(-)** keys. If **FEED INTERVAL** is on display, clear feeding interval by pressing **FEED INT** to display feeding interval, then press **CLEAR** key.

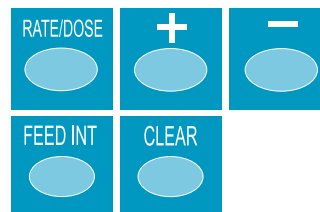


STEP 4 – Continuous Feeding/Infinite Dose

If **FEED INTERVAL** is on display, clear feeding interval by pressing **FEED INT** to display feeding interval, then press **CLEAR** key. To program an infinite dose, press **RATE/DOSE** key twice to display **DOSE** and hold the **(+)** key until display reads INF.

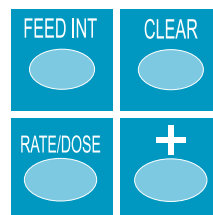

STEP 4 – Intermittent/Interval Feeding

Press **RATE/DOSE** key to display dose. Program the prescribed dose using the **(+)** and **(-)** keys. Press the **FEED INT** key to display feeding interval. Program the desired number of hours and minutes using the **(+)** and **(-)** keys.

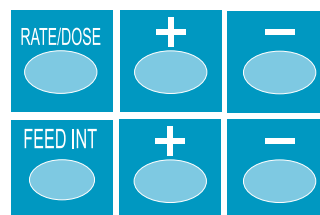

STEP 5

Press the **RUN/PAUSE** key to start the pump.



STEP 6

When feeding is complete, press the **ON/OFF** key for 1.5 seconds to turn the pump off.



Clearing settings

Press **CLEAR** key to reset displayed function to its minimum value. Press and hold for three seconds to clear rate, dose, feed interval and volume delivered to minimum values at the same time.



Viewing volume delivered

Press **VOL/TOTAL** key once to display volume delivered for current feeding. Press key again to display total cumulative volume delivered in all feed cycles since total volume was last cleared.

